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EN INSTRUCTION FOR USE

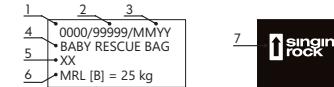
Baby rescue bag (BRB) is designed for rescue transporting of the children of a height of 40–110 cm. Recommended angle of the BRB should be chosen with regards to current health condition. The maximum rated load is 25 kg. The child must be fixing by 3 belts with Velcro and with the safety straps led to cross during the transporting. Fixations with both parts have to be done with regards to the free breathing and possible movement of the child. It is necessary to prevent e.g. positional asphyxia (suffocation) due to bad fixation.

For transporting by the helicopter is necessary to close both parts of cover (top and bottom) because of the strong flow of air from a helicopter rotor. The stabilization strap of the top cover is necessary to set and tighten optimally to prevent the camber of the cover to the face of the child during the hanging of the BRB.

It is not necessary to use the top cover and the fixation strap during the transporting without the helicopter. The position of the BRB is intended by the rescuer with regards to current health condition of the child. Adjusting the position of the BRB is done by establishing of the carabiner into the same coloredloops.

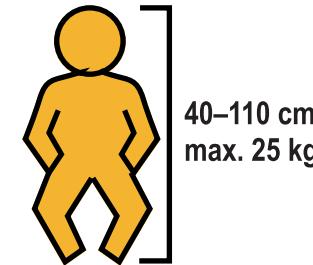
It is important to let free the sides ventilation holes during the transportation of the child. The child must be check visually during the transportation.

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